The SleepPro anti-snoring Chin Support Strap wraps around the chin and the top/ back or the head to encourage the jaw to stay closed and mouth shut during sleep.

It's a simple device, easy to use, and easy to keep clean. Our exclusive one-piece, two-strap design ensures comfort and position during sleep.

The SleepPro anti-snoring Chin Support Strap:

- A simple, affordable anti-snoring device.
- Fits easily across the chin and around the top or back of the head.
- Comfortable to wear and brings instant results.
- The usual treatment for open mouthed snoring.
- Reduces the problem of dry mouth during sleep.
- Easy to clean using a hand or a machine wash.
- Affordable, long lasting and fits all sizes.

Tip: Use the anti-snoring chin strap in combination with a SleepPro mouthpiece or nasal dilator. For example, the chin strap can help to keep the mouthpiece in place during sleep.

MEDITAS Ltd.
PO Box 6422
Milton Keynes, MK10 1JY
United Kingdom
0203 302 6269
info@sleeppro.com
www.sleeppro.com

SleepPro Europe
Postbus 995
1185 AZ Amstelveen
Nederland
info@sleeppro.eu
www.sleeppro.eu

UK SleepPro Chin Strap

Fits all sizes easily and is comfortable to wear

- The SleepPro anti-snore Chin Support Strap is a specialised treatment for open mouthed snoring and is a well proven solution to minimise the problem of dryness of mouth during sleep.
- With the chin strap on, you can sleep peacefully and without any unnecessary discomfort or pain.
- This anti-snore chin strap is designed for easy use and has an easily adjustable, sleek design.

Simple and easy to use and fits in seconds

- Wrapping around the chin and top/back of head, the chin strap ensures that the mouth of the person snoring remains closed all night without it coming off. The exclusive one-piece, two strap design ensures comfort, whilst firmly remaining in the correct position throughout the duration of your sleep.
- The chin support strap is affordable, long-lasting and after a few nights of using it most snorers report that do not even notice wearing it. This device offers an instantaneous and non-invasive remedy for snorers, and with it's fully adjustable function it can be worn by anyone of any age.

Please read this instructions carefully before fitting your new chin support strap

Position the Chin Support Strap as shown in the pictures. For additional closing force, place straps more toward the front, even to the extent or overlapping the rear strap onto the front strap. You can cross the rear strap over the front strap for more upward force.

Experiment with differing tension, using our exclusive two-strap adjustable tension and positioning system. Once a comfortable workable position is obtained, you can simply slip the unit off and on without changing the adjustment positions. It can also be repositioned at any later time, if desired.

The adjustable two-strap unit ensures a comfortable level of tension while ensuring position integrity during sleep. Note which strap you prefer as the rear or front by remembering in which position you place the labeled strap.

Fitting instructions

- 1 Position the device over the chin and the top/back of head (front and rear strap respectively), as shown above.
- 2 To alter the upward force, place the straps towards the front of the head. They can be overlapped for increased tension, and also moved towards the back to reduce it.
- 3 The exclusive two-strap 'tension and positioning' system helps to achieve the desired upward force.

Once obtained, the chin strap can be removed and used again and again without having to adjust it each time, but it can be repositioned at any time if necessary.

Washing instructions

- Hand or machine wash in water below 40 °C
- Mild detergent may be used
- Do not tumble-dry
- Do not dry clean
- Do not store in temperatures exceeding 40 °C

Material: Neoprene

Disclaimer: While it is unlikely you will encounter any significant problems with the use of a simple SleepPro Chin Support Strap, please note that MEDITAS Ltd. accepts no liability for problems arising from the use of SleepPro products. If you do experience any complications, then please stop using the item immediately and contact us. Please note that the conditions of sale and use of this product relate to the laws of England and Wales and the regulatory guidelines of the United Kingdom.

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